

Duo-Scan 3

Innovative Imaging



Quick Instruction Guide


The Buttons



Scrolls through preset Modes.



Switches between Power or Gain adjustments.

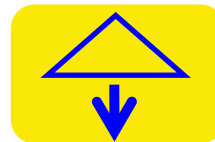

Power or Gain down.




Power or Gain up.



Short Press Freezes Image.
Long Press Switches On & Off.

Menu is displayed for approx 6 seconds then disappears, pressing any button switches the menu back on.

Battery Meter

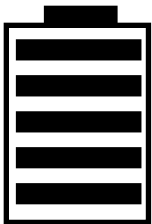

Each bar on the battery meter represents approximately 1 hour of capacity.

The battery meter will flash when the remaining capacity is down to the last 5 minutes.

BCF Technology Ltd
3 Tailend Court
Starlaw Road
Livingston EH54 8TE

Email: info@bcftechnology.com
Web: www.bcftechnology.com
Tel: +44(0)1506 460023
Fax: +44(0)1506 460045

